EN Directions for use
NL Gebruiksaanwijzing
FR Mode d'emploi
DE Gebrauchsanweisung

■ **EN Caution:** not all articles mentioned below are part of your first aid kit. Please check the content on the enclosed table of contents.

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Intended application

For cleaning and covering surface wounds.

For non-professional use

Sterile compress for external use.

Operation

The compress covers the wound and absorbs discharge.

Directions for use

- Carefully clean the wound and surrounding area.
- Remove the sterile packaging and place the compress over the wound, double-folded if required. Do not touch the side that comes into contact with the wound.
- Attach the compress using a bandage or adhesive plaster.
- Check the wound regularly and replace the compress at least daily.

⚠ Do not use the compress if it is dirty, or if the sterile packaging has been damaged or previously opened in error. Stop using in the event of skin irritation or infection and consult a doctor if it persists. Do not reuse the compress as this can cause infection.

Report serious incidents to the manufacturer and the competent authority.

■ **NL Steriel niet-verklevend kompres 10x10 cm / Steriel nonwoven kompres 5x5 cm – 7,5x7,5 cm – 10x10 cm**

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■ **EN Wound closure strips 4x76 mm**

Description

Flexible wound closure strips made of polyamide fabric, non-elastic backing. They are coated with a pressure-sensitive, hypoallergenic adhesive. Can be used on contoured body areas or where bending and movement occur.

Intended Use

- Adhesive wound closure strips are used in the treatment of lacerations and surgical incisions.
- Can be used in combination with skin sutures and staples or after their removal for wound support.

Contraindications

- Don't use on patients with known sensitivity to acrylic adhesive.
- Don't use on infected wounds.
- Don't use in case of high tension wounds which cannot be easily approximated with fingers or forceps. Skin shearing, skin blistering, or loss of adhesion may result if excessive tension is applied.

Precautions

- Ensure that the skin is clean, dry, and free of skin oils to assure good adhesion.
- The occurrence of postoperative edema may cause skin shearing, skin blistering, or loss of tape adhesion to occur at either end of the strip.
- Upon removal of the strip, skin peeling may occur as a result of the usage of any surgical tape or adhesive.
- Rarely, hypopigmentation or hyperpigmentation may occur due to the application of adhesives to the skin.
- Mild acne and folliculitis have been observed occasionally in testing on healthy volunteers.

Instructions for Use

- Open package. Remove card end tab.
- Carefully line up the edges of the wound.
- Starting at the middle of the wound, place half of the strip on one side of the wound, gently bring the other side of the wound towards it, and then pass the strip over.
- Place strips alternately above and below the first strip – this helps to match up the edges and keeps the skin tension equal. Continue this process until you've completely closed the wound.
- For additional fixation, to reduce tension and to anchor the rows of strips in place, put two strips vertically across the rows – one on each side of the wound.

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EN Sterile saline wipes 6x6 cm

0.9% sodium chloride solution. Use sterile saline wipes for cleaning surface wounds and surrounding skin.

Do not reuse saline wipes as this can cause infection. Do not use to cleanse deep wounds or wound cavities.

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EN Sterile quick bandage 6x8 cm - 8x10 cm - 10x12 cm

Intended application For rapid covering and binding of small, open, bleeding wounds.

For non-professional use Sterile quick bandage for external use.

Operation The quick bandage covers wounds, absorbs moisture, and stops bleeding by means of pressure.

- Instructions for use 1. Carefully clean the wound and surrounding skin. 2. Remove the sterile packaging and place the compress over the wound.

Do not use the quick bandage if it is contaminated or if the sterile packaging is damaged or has been opened in error.

Report serious incidents to the manufacturer and the competent authorities.

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EN Non-woven dressing 60x80 cm

Intended application To cover large wounds, sterile and pressure-free.

For non-professional use Sterile dressing for external use.

Operation The dressing covers the wound and absorbs wound fluid.

Instructions for use Remove the sterile packaging and place the dressing over the wound. Do not touch the surface that comes into contact with the wound.

Do not use the sterile dressing if it is unclear or if the sterile packaging is damaged or has been opened in error.

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Service Best International B.V. P.O. Box 76 | 5500 AB Veldhoven | The Netherlands | www.carpoin.nl